
INSTRUCTIONS FOR USE of all Sontec Instruments Whisk'R Brushes

IMPORTANT

Intended Use Statement

Intended to remove organic residue from surgical instruments after surgical procedures.

NOT FOR USE ON HUMANS

Instructions for Use

1. Inspect brush between uses and **replace** if worn, frayed, bent, or otherwise damaged.
2. **Inspection is the responsibility of the user.**
3. Worn bristles are ineffective in cleaning and could damage the instrument.
4. Ensure brush size is adequate for cleaning specific device channels.
5. To remove gross soil, flush the device with water while using the brush through the device channel until the soil is removed.
6. For specific device cleaning instructions, refer to the device manufacturer's IFU.

Sontec Instruments are guaranteed to be free of defects in materials and workmanship. Any Sontec Instrument that is determined to be defective will be replaced or repaired at our discretion. Instruments not properly cared for, used for purposes not intended, or used beyond the design capacity will NOT be covered. Our liability under this guarantee is limited to the repair or replacement of the defective merchandise.